



Strataderm®

scar therapy gel

Stratamed®

advanced film-forming  
wound dressing

Stratacel®

advanced dressing  
for fractional procedures

Advanced wound healing,  
post-procedure care and  
professional scar management

 **Stratapharma**  
Switzerland

product description



**Strataderm is a once per day professional scar therapy gel.**

Strataderm is a robust and durable formulation allowing for once a day application.

**Strataderm application**  
on old/new scars



**Stratamed is an advanced film forming wound dressing.**

Stratamed was developed with a higher viscosity to ensure that it remains in stronger contact with the wound bed, suture line or damaged surface.

**Stratamed is generally used following full field procedures, after surgery and deeper wounds.**

**Stratamed application:**

1. post full field ablative resurfacing
2. on sutured wound
3. on a deeper wound



**Stratracel is an advanced dressing for fractional procedures.**

Stratracel was developed with a lower viscosity so that it is easier to spread and remove.

**Stratracel is generally used following post fractional procedures**

**Stratracel application:**



features & benefits

- Soften and flattens raised scars
- Relieves itching and discomfort of scars
- Reduces redness and discoloration
- Once daily application (or twice daily to exposed areas or as advised by the physician)
- Can be used in combination with other more invasive therapies to enhance overall efficacy
- Rapidly drying, non-sticky, transparent, can be covered by sunscreen and cosmetics

**Stratamed** allows scar management to begin immediately post procedures (before suture removal).

**Common features & benefits for Stratracel and Stratamed**

**Flexible, full contact**  
Stratamed and Stratracel dry to form a thin and flexible protective film that ensures constant contact with the damaged skin.

**Reduce downtime**  
Stratamed and Stratracel reduce downtime by creating an optimal environment for faster healing.

**Antimicrobial**  
Stratamed and Stratracel are bacteriostatic; they protect from microbial invasion whilst reducing the risk for contact dermatitis.

**Relieve symptoms**  
Stratamed and Stratracel provide symptomatic relief from itching, burning and discomfort as well as reducing redness and swelling.

**Non-reactive**  
Stratamed and Stratracel are inert, have no measurable pH, contain no alcohol, parabens or fragrances, making them suitable for children and people with sensitive skin.

**Medical device**  
Stratamed and Stratracel are FDA listed Class I medical devices (wound dressings). Suitable for use on open wounds, surgical incisions and mucosa (i.e. nasal, oral and vaginal).

**Stratracel** was designed to adhere softly to existing epidermis making it the ideal post procedure dressing for fractional skin injuries.

indications/procedures

**OLD AND NEW SCARS**

- Strataderm has been developed for use on **all types of scars:**
- Old and new scars
  - Hypertrophic scars
  - Keloids
  - Red scars
  - Elevated scars
  - Pruritic (itchy) scars

**scar management**

**FULL FIELD PROCEDURES, SURGERIES AND DEEPER WOUNDS**

**cosmetic and medical procedures**

**POST FRACTIONAL PROCEDURES**

- Cosmetic indication/procedures**
- Laser procedures (full field)
  - Chemical peel (full thickness)
  - Tattoo removal (fully ablative)
  - Scar revision (blepharoplasty, lifts, rhinoplasty)
  - Cosmetic surgery body (breast augmentation/reduction, lifts, abdominoplasty)
- Medical indication/procedures**
- Surgical excisions and biopsies
  - ED & C's
  - Other surgery over sutures (i.e. Mohs)
  - Healing by secondary intention (i.e. wound dehiscence)
  - Chronic wounds (i.e. pressure & diabetic wounds)
  - 2<sup>nd</sup> degree burns (without epidermis)
  - Accidents and trauma
  - Dermabrasion

- Cosmetic indication/procedures**
- Laser procedures (fractional)
  - Chemical peel (superficial)
  - Tattoo removal (fractional)
  - Microblading
  - Microneedling
  - Radiofrequency
  - Vaginal rejuvenation
  - Other fractional devices
- Medical indication/procedures**
- Cryofreezing/actinic keratosis
  - Post-5-FU therapy
  - Medical tattooing (i.e. post breast reconstruction)
  - Inflammatory reactions (i.e. dermatitis)
  - 1<sup>st</sup> and 2<sup>nd</sup> degree burns (with epidermis)

clinical results

**Strataderm®**  
scar therapy gel




Before treatment After 2 months treatment  
Left scar remained untreated, right scar treated for 2 months

**Linear hypertrophic scar after nevis extraction!**  
Age of scars before treatment: 14 months  
Monotherapy treatment with Strataderm

**Stratamed®**  
advanced film-forming wound dressing



**Mohs surgery reconstruction!**  
Monotherapy treatment with Stratamed

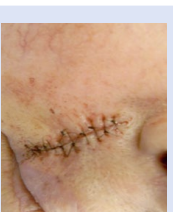

Before treatment Month 2

**Stratracel®**  
advanced dressing for fractional procedures

**Laser resurfacing (acne scars)?**  
Treatment following thermage and CO<sub>2</sub> fractional laser

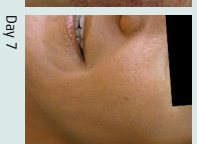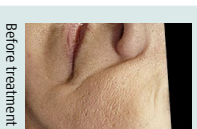

Before treatment During treatment Day 7

References: 1. Data on file, Strathpharma AG; 2. Sandhofer M, Schauer P, Stamm 2012; 106:1-7



[us.strataderm.com](http://us.strataderm.com)



CE Class I Medical Device  
FDA Listed

**Available tube sizes**

0.17 oz	0.35 oz	0.70 oz	1.75 oz
5 g	10 g	20 g	50 g

[us.stratamed.com](http://us.stratamed.com)



CE Class I Medical Device  
FDA Listed

**Available tube sizes**

0.17 oz	0.35 oz	0.70 oz	1.75 oz
5 g	10 g	20 g	50 g

[us.stratacel.com](http://us.stratacel.com)



CE Class I Medical Device  
FDA Listed

**Available tube sizes**

0.35 oz	0.70 oz	1.75 oz
10 g	20 g	50 g

Manufactured by: Stratpharma AG, Aeschenvorstadt 57, CH-4051 Basel, Switzerland  
 Represented in the USA by: Stratpharma Inc., 1455 Frazee Road, Suite 500, San Diego, CA 92108, USA

